

# ONEcleaner Enzyme Detergent Lubricant Cleaner

Enzyme Surgical Instrument Detergent Lubricant Cleaners **clean faster**.  
The **ONEcleaner** Enzymatic Surgical Instrument Detergents **cut costs**.  
Lubricating Enzyme Surgical Instrument Detergents clean **residue free**.

0225-32 easy FOAM-it soaking ONEcleaner enzyme detergent lubricating cleaner, 12 ea. 32 oz.  
0225-01 ONEcleaner enzyme detergent lubricant cleaner, 4 gallons, with 1 pump  
0225-05 ONEcleaner enzyme detergent lubricant cleaner, 5 gallons jug  
0225-15 ONEcleaner enzyme detergent lubricant cleaner, 15 gallons jug  
0225-55 ONEcleaner enzyme detergent lubricant cleaner, 15 gallons jug



The **ONEcleaner** delivers enzymes with detergents, that work together, during every cleaning cycle, to rapidly break down all forms of surgical contamination, stains, and mineral encrustation.

- four Enzyme cleaners; protease, amylase, carbohydrase, and lipase
- Detergent surface cleaners
- Lubricants that are water soluble and free rinsing
- **neutral pH** Surface Conditioners that strengthen the protective passive layer of stainless steel

The highly concentrated **ONEcleaner** delivers an average of 400% more cleaning power. One gallon can deliver 512 gallons (1938 liters) of cleaning power. The dilution rate is .25 to .5 ounces per gallon diluent (1.9 ml liters to 3.8 ml per liter) depending on the level of bioburden, encrustation and stains. These recommended dilution rates deliver *highly concentrated* ingredients.



Customer Service: 1-509-220-2456, email: [customerservice@ergo-logistics.com](mailto:customerservice@ergo-logistics.com)  
Technical Support: 1-509-747-5027, email: [support@ergo-logistics.com](mailto:support@ergo-logistics.com)